
**Directions for Use (Single Patient Use Only) • Mode d'emploi (utilisation pour patient unique seulement) • Gebrauchsanweisung (nur bei einem einzigen Patienten anwenden) • Instrucciones de uso (únicamente para el uso de un solo paciente) • Istruzioni per l'uso (esclusivamente monopaziente) • Instruções de utilização (apenas para utilização por um único doente) • 使用説明 (僅供一位患者使用) • 使用説明書 (患者一人用)**

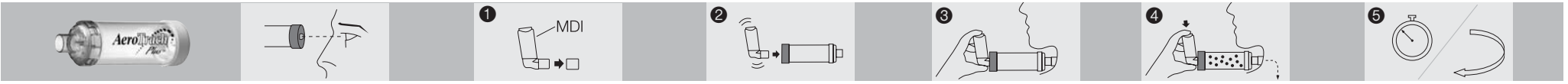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Manufactured by:  
**Trudell Medical International\***  
725 Third Street, London, Ontario,  
Canada N5V 5G4  
☎ +1 (519) 455-7060  
☎ +1 (519) 455-7858  
✉ customerservice@trudellmed.com  
[www.trudellmed.com](http://www.trudellmed.com)

**Authorised European Representative:**  
Trudell Medical International Europe Limited  
Bicocity Nottingham, Pennyfoot Street,  
Nottingham, United Kingdom NG1 1GF  
REGISTERED  
ISO 13485  
QUALITY SYSTEM

**AeroTrach Plus\***  
Anti-Static Valved Holding Chamber

Printed in Canada. 101061-001 Rev F. \* trade marks and registered trade marks of Trudell Medical International. Vital-Oxide® is a trade mark of Vital Technologies. Copyright © 1999, 2000, 2007 – 2009 Trudell Medical International. All Rights Reserved. Patents pending.

**WARNING:** Remove device if patient shows signs of difficulty.

**Intended Use:** The **AeroTrach Plus\*** VHC is intended to be used by patients who are under the care or treatment of a health care professional or physician. The device is intended to be used by these patients to administer aerosolized medication from most MDIs prescribed by a health care professional or physician. The intended environments for use include the home, hospitals and clinics.

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- Cautions** ENG
- **AeroTrach Plus**® VHC is designed for use with 15mm tracheostomy tubes with spontaneously breathing patients only.
  - This device is designed as a slip fit onto the tracheostomy tube connection. It must be held in place during administration of medication.
  - Do not spray more than ONE PUFF into the chamber at a time.

- Notes**
- VHC = Valved Holding Chamber
  - Ensure directions for use have been read prior to use and are kept available at all times.
  - If you have questions about the performance or usability of this product, please contact your healthcare provider.
  - Do not leave the chamber unattended with children.
  - Do not clean in dishwasher, boil or autoclave.
  - This product contains no latex.
  - Product should be replaced after 3 – 4 months of use. Environmental conditions, storage and frequency of use can affect product life span.
  - When not in use, store in suitable container.
  - Do not share.
  - This device does not deliver supplemental oxygen.

- Instructions for Disinfection**
- After the clean water rinse, the following may be safely used: Enzyme Solvents, Vital-Oxide®, 1 part bleach/10 parts water solution. These should be soaked for 30 minutes followed by a clean water rinse.

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